CE
0499

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PROFESSIONAL USE ONLY

- Trifaceted needles with the bevel face indicated and thin walls which provide improved flow of the anesthetic solution.
- Sealed by melting weld and super safe protectors.
- Non pyrogenic.
- Sterilized by ethylene oxide.

Composition: Made of special high technology stainless steel, they are siliconized to soften the penetration of the cannula into tissues.

Indications: For injection of anesthetics, according to the techniques advocated for these needle sizes.

Presentations: Box containing 100 sterilized disposable needles (Size: 30 G Short).
Box containing 100 sterilized disposable needles (Size: 30 G Xshort).
Box containing 100 sterilized disposable needles (Size: 27 G Long).

STERILE PRODUCT.

THE SEAL GUARANTEES THE STERILITY OF THE NEEDLE.

THE NEEDLES ARE INTENDED FOR SINGLE USE AND SHOULD BE DISPOSED AFTER USE.

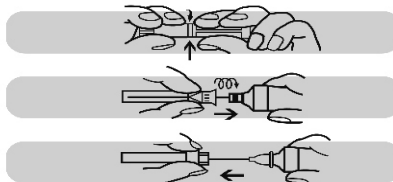
Storage and Conservation: Keep the product in its original well closed package and store in a dry place.

Shelf-life: 5 years.

Warnings and Precautions:

- Do not use if package is damaged, open or wet.
 - Do not use if seal is broken.
 - During use, do not change needle direction while it is inside tissue.
 - Never force the needle if there is any resistance.
 - The needles should be disposed and destroyed immediately after use to avoid injuries or reuse by non authorized people.
- Do not dispose without first isolate the needle with the cartridge case. Dispose preferably in special containers aimed for contaminated materials.
- Since they are used, needles should not be re-sterilized or reused.
 - This product should be used by dentists only and just for the purposes described in these Instructions for Use.

Way to use: Rotate the edges of the cartridge case until the seal is broken. Do not use product if the seal is already broken. Remove the smaller cap. Screw the needle on the syringe and remove the protection.

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INDÚSTRIA E COMÉRCIO S.A.