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2a dobra

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English

PROFESSIONAL USE ONLY

NATURAL BOND PLUS is composed of a single component light cured bonding agent (DE PLUS) for enamel and dentin and an activator for dual cure (ATV).

NATURAL BOND PLUS promotes adhesion in direct and indirect procedures and is compatible with all materials for dental restoration, as conditioners, composite restorative components and resin cements. It contains fluoride. The mixture of **NATURAL BOND DE PLUS** with **NATURAL BOND ATV** forms a dual cure adhesive i.e. light curing and chemical curing or self-healing.

Composition:

NATURAL BOND DE PLUS contains: Resin PMGDM 50%, Resin 2 HEMA 97 %, TEGDMA, Camphoroquinone, Resin 4 EDAMB, PHFA, 2,6 DI, Bis-GMA, Neutral Hydrated Alcohol.

NATURAL BOND ATV contains: Sodium NTG-GMA and Ethanol.

Alpha Etch contains: Phosphoric Acid, Colloidal Silicon Dioxide (Aerosil 200), Methylene blue (1% Solution) CI52015, Bi distilled Glycerin and Deionized Water.

Indications:

NATURAL BOND DE PLUS is indicated to bond any composite and compomer to the enamel or dentine, for both direct and indirect restorations. For adhesive cementation of ceramic indirect restorations, ceromer or composite, like veneers, inlays, on lays and crowns (only if associated with resinous cement).

Presentations:

Box containing: 1 bottle (4 mL) of **NATURAL BOND DE PLUS** + 1 bottle (3 mL) of **NATURAL BOND ATV** + 1 syringe 3g (2,5 mL) of ALPHA ETCH + accessories (applicator tips).

Box containing: 1 bottle (4 mL) of **NATURAL BOND DE PLUS** + 1 bottle (3 mL) of **NATURAL BOND ATV**.

Box containing: 2 bottles (4 mL, each) of **NATURAL BOND DE PLUS** + 1 syringe 3g (2,5 mL) of ALPHA ETCH + accessories (applicator tips).

For replacement

Box containing: 1 bottle (4 mL) of de **NATURAL BOND DE PLUS**.

Box containing: 1 bottle (3 mL) of **NATURAL BOND ATV**.

Box containing: 2 bottles (4 mL, each) of **NATURAL BOND DE PLUS**.

Storage and Conservation:

NATURAL BOND PLUS and Alpha Etch Must be kept away from excessive heat (avoid temperatures higher than 40°C). Do not expose to excessive heat. Keep the bottles tightly closed. Under these conditions, the product is valid for up to 2 years as of the manufacturing date. Do not use this product if shelf life had expired.

Instructions for Use:

The use of gloves is recommended during the procedure.

Attention: The patient must be informed on the risk to occur sensitivity in the place of the procedure. Use absolute isolation whenever it's possible. Use, when it's close to the pulp, calcium hydroxide and/or ionomer glass cement.

Dosage:

1 - Procedures for direct restoration: Drip a drop of **NATURAL BOND DE PLUS** in a Dappen pot immediately before utilization.

2 - Procedures for indirect restoration with dual cure, using the **NATURAL BOND ATV** activator: Mix in a mixing tray the amount of drops needed for the restoration, in proportion to 2 drops of **NATURAL BOND DE PLUS** for each drop of **NATURAL BOND ATV**.

Product wastes must not be reused. They must be disposed as the local law in force.

Direct Bonding of Composite to Dentine/Enamel Procedures (**NATURAL BOND DE PLUS**):

1 - Isolate, clean and prepare the tooth for a restoration.

2 - Etch dentine for 30 seconds and enamel for 15 seconds with Alpha Etch.

3 - Rinse with water spray.

4 - Remove the excess of water with a jet of air, avoiding dentine dehydration. Moisture should be visible on the surface (shiny dentine), but should not form puddles, which should be removed with brush or absorbent paper. Avoid saliva contamination.

5 - Place 2 drops of **NATURAL BOND DE PLUS** in a mixing well. Place the first coat, scrub for 5 seconds, wait 20 seconds, gently apply an air jet for 5 to 10 seconds to remove the solvent and then apply a second coat. Avoid too thin coats, generated by the use of excessive air.

Wait 20 seconds.

6 - Photoactivate for 20 seconds. The surface should appear glossy; if not, repeat the procedure.

7 - Proceed with the placement of the increments of resin.

Dual Cure Procedures (**NATURAL BOND DE PLUS** + ATV):

The dual cure capability of **NATURAL BOND DE PLUS** + ATV greatly simplifies the use of the product when placing indirect restorations that cannot be light cured. Whenever a dual cure adhesive is desired, use the wet bonding technique and follow the instructions below:

1. Isolate, clean and prepare the tooth for a restoration.

2. Prepare the mating surface of the indirect restoration for bonding (fluoridric acid, silane or jet with aluminum oxide, depending on the material).

3. Etch dentine for 30 seconds and enamel for 15 seconds with Alpha Etch.

4. Rinse with water spray.

5. Remove the excess of water with a jet of air, avoiding dentine dehydration. Moisture should be visible on the surface (shiny dentine), but should not form a pool, that should be removed with brush or absorbent paper. Avoid saliva contamination.

6. Attention: The bottle of **NATURAL BOND ATV** should be shaken immediately before the use.

7. Place 2 drops of **NATURAL BOND DE PLUS** and 1 drop of **NATURAL BOND ATV** in a mixing well.

Place 2-3 coats of this combination on the moist dentine and enamel, scrub for 5

seconds, wait 20 seconds between coats to allow solvent to evaporate. After the final coat, wait 20 seconds, dry the surface for a few seconds with air. The surface should appear glossy; if not, apply additional coats until glossy surface appears.

8. Apply the combination to the mating surface of the indirect restoration.

9. For indirect restorations requiring self-cure or dual cure capabilities, or whenever dual cure capability is desired, do not photoactivate this adhesive layer.

10. Proceed as usual with the instructions for cementation or restoration placement.

Photoactivate in the end of the restoration.

Precautions:

Avoid the use of **NATURAL BOND DE PLUS** + ATV in patients with previous history of allergy to compounds containing acrylate. In the case of allergy, you must discontinue the use immediately and advise the patient to see a doctor. Do not allow contact with soft tissues or the skin. In the case of contact, you should immediately remove the material with hydrophilic cotton soaked with water and rinse thoroughly with water. Avoid contact with the eyes; in case of accidental contact, wash the eyes immediately with water and see a doctor. This product must be used only by dentists and for the intended use described in the Instructions for Use.

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DFL Indústria e Comércio S.A.